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ANGELSOUNDS JPD-100S MINI HEARTBEAT ULTRASOUND FRUIT

User manual

Please read the manual before first use. Keep the manual for future reference. Also, carefully read the warnings and cautions below before use.

Use

The device detects the fetal heart rate/pulse. After connecting the headphones, the fetal heartbeat sound can be heard and counted. The device is usually used until the 16th week of pregnancy or later, depending on the progress of the pregnancy. The device detects fetal life from the beginning of pregnancy until delivery and can be a general indicator of fetal well-being. Normal fetal heart rate range: 110 to 160 beats per minute.

Description

This is a lightweight, portable detector. It provides advanced detection functions and a full range of fetal heartbeat sound. The device is mainly used to detect fetal heart rate (FHR) and fetal heartbeat sound (SFH).

By examining these indicators, the growth and development of the fetus can be determined. It is applicable to the gynecological-obstetric department and clinic. In accordance with the classification criteria set out in Annex IX of the Medical Devices Directive 93/42/EEC, the product belongs to Class IIa based on Rule 10, "Devices for direct diagnosis or detection of physiological processes". The device is powered by a 9 V battery (battery not included).

Operating principles

The device consists of a transmitting unit, a receiving unit, a signal processing unit and a signal output unit (e.g. headphones).

Ultrasound waves are transmitted from the piezoelectric ceramic at the front of the probe into the uterus of the pregnant woman. The echo/reflected wave is received by a second piezoelectric ceramic at the front of the probe. Information is then detected and demodulated from the received signal. The frequency corresponds to the rhythm of fetal systole and diastole. After the heart valves oscillate, a Doppler frequency deviation is created. The output signal of the heart valve vibrations is transmitted and sent to the signal demodulation unit, thereby obtaining a Doppler frequency signal, which is then processed into an audio signal audible through headphones.

Contraindication

In normal cases, there are no contraindications. Please consult your doctor for individual cases.

Undesirable effect

No adverse effect was found.

Note: The

device is not intended as a professional medical device and does not replace diagnostic equipment in hospitals. If the device does not detect the location of the heart or record the heartbeat or pulse rate, See your doctor immediately. If you do not feel any fetal movement, also see your doctor immediately.

Safety notice

Before using for the first time, please pay attention to the safety instructions and warnings. Follow the instructions given. Otherwise, there is a risk of personal injury or damage to the device.

Danger

Fire and explosion hazard – do not operate the device in an area where there is a risk of explosion, explosive or flammable substances or open flames.

Warning

1. Only use batteries, gels, cables and devices verified by Jumper Medical Ultrasonic Instrument Co., Ltd. Otherwise, there is a risk that the device will not function properly.
2. Make sure that there are no other devices nearby that could cause interference and influence while using the device.
3. The manufacturer recommends that ultrasound exposure be as low as possible. Follow the instructions and proceed in a way that minimizes exposure.
4. Aid for healthcare professionals. The device should not be used in place of standard fetal monitoring equipment in hospitals. The device is not a substitute for professional medical examination.
5. Protect the device from extreme high/low temperatures, high humidity and extreme pressures. Otherwise, there is a risk of damage to the device or personal injury.
6. Do not disassemble or tamper with the battery. Recycle the battery according to the applicable standards and laws of the country. Do not dispose of the device or battery in a fire, otherwise there is a risk of explosion.
7. Do not use the device near radio frequency devices such as mobile phones, CB radios, FM2 radios. Otherwise, there is a risk of interference between the devices. The device may not work properly. Turn off the RF devices in the vicinity before using this device.
8. Connected devices must comply with certified IEC standards.
9. Do not use oxidizing detergents for cleaning. Use disinfectants based on ammonium salts or glutaraldehyde. Otherwise, there is a risk of damage to the metal parts.
10. The device is intended for indoor use only. Do not use the device outdoors.

11. Store the device in a dry and clean place. Store the device in suitable temperatures. Before use, leave the device at room temperature before using it.
12. Recycle the device according to the applicable standards and laws of the country.
13. The device is not a toy. Keep the device out of the reach of children.

First use


Before first use, make sure that the device has arrived in good condition and in the packaging. has no visual damage. Make sure that the individual parts are The components are also in good condition and not visually damaged.

Package contents

1x device, 1x user manual, 1x headphones

Buttons and LED indicator

1. Volume on/off wheel
2. LED on/off indicator
3. Headphone port
4. Probe
5. Battery compartment
6. Audio output port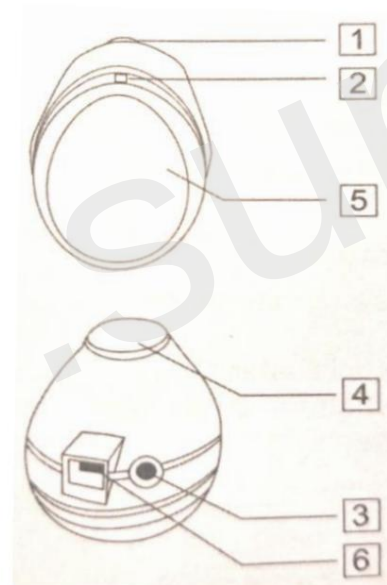

On/Off/Volume Wheel: Turn the device on/off, control the volume.


The device has one headphone port and one USB interface as an audio output port. The USB interface is used to connect to a data cable.

Battery

The device is powered by a 9 V alkaline battery (IEC standard 6F22 or equivalent).

Control

Open the battery compartment and insert a 9V lithium battery. Make sure the polarity is correct. Then close the battery compartment.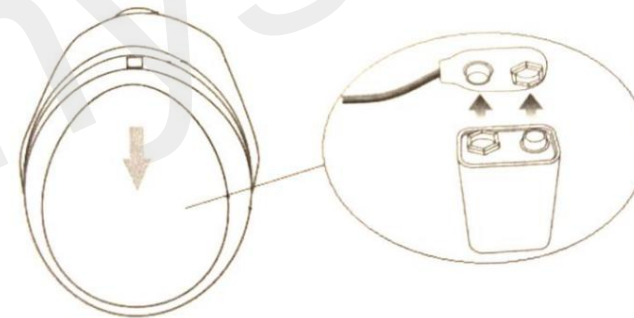


Note: Open the battery compartment cover in the direction of the arrow. If the device will not be used for a long time, remove the battery and store it in a dry and dust-free place.

Connecting headphones

Before use, install the headphones or headset. Plug the headphones into the device.

Use/control

1. Turn the on/off wheel to turn on the device.
2. Move clothes away from your pregnant belly.
3. Apply a little gel to the affected area.
4. Find the fetal position with your hands. Estimate the position of the heart as shown in the picture below. Place the probe at the given location and at an angle so that you can hear the sound of the heartbeat. Generally, the fetal heart position is 1/3 lower than the umbilicus in the early stages, and then moves up as the gestational period increases. The fetal heart position varies from woman to woman (slight deviation to the left or right). Make sure that the entire surface of the probe is in contact with the skin. The sound is clearly audible and clear. If necessary, add or remove gel on the skin.

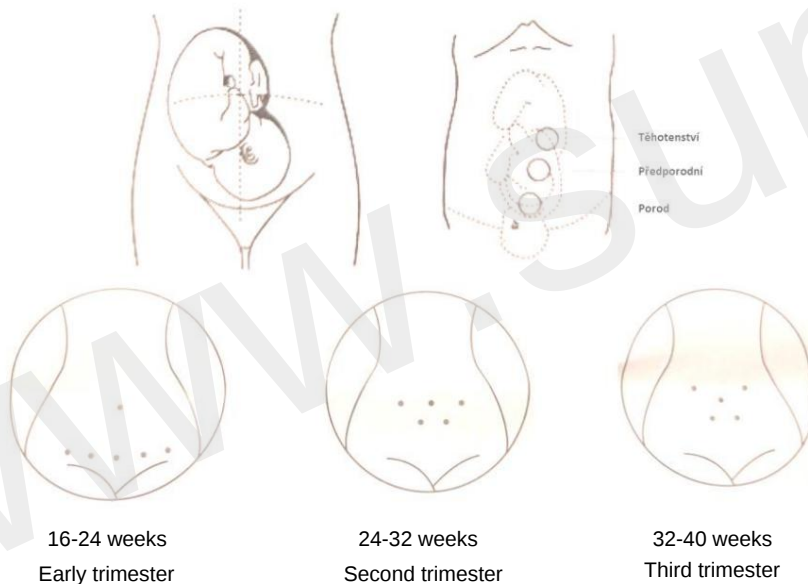Heart rate calculation

Count the fetal heart sounds in one minute. The number you get is the fetal heart rate. Repeat the count at least 3 times.

Recording

After capturing the fetal heartbeat sound, one port can be connected to headphones and the other to a computer/mobile phone. The recorded audio files can be played back at any time.

Maintenance

Be careful when wiping and cleaning. Otherwise, there is a risk of damaging the device. The probe is a sensitive part of the device. Be careful when cleaning the probe. After use, wipe off any remaining gel with a dry cloth or tissue. After cleaning the device, always check it for damage. If you notice any visual changes to the device, battery or accessories (damaged ports, bent cable, pins), stop using the device immediately. Contact a service center.

Store the device at an appropriate temperature. The device does not require calibration.

Do not use corrosive liquids and detergents or solvents and other chemicals, such as acetone-based ones, for cleaning. Do not use aggressive disinfectants or mix disinfectants.

Wipe the device only from the outside. Protect the electrical components from water and high humidity (device protection level IPX4). Do not immerse the device in water. Before cleaning, switch off the device and disconnect it from all other devices or cables. Protect the device from dust. If the device is very dirty, a damp cloth with a little soap can be used. Then dry the device immediately. Protect the device probe from scratches. Do not sterilize the device (regardless of the sterilization temperature).

Protect the device from abrasive gases. Protect the device from metal objects. The device can be wiped with 70% ethanol. Dry the device immediately after cleaning.

Specifications

Ultrasonic wave emitting frequency: 3 MHz
Sensitivity up to 200 mm from the probe: Doppler frequency 500±50 Hz, target speed: 10 cm/s-40 cm/s±90 dB
Room acoustic pressure: $\dot{y}$ 1 MPa
Output: <20mW
Effective area of the ultrasound probe: 2.65 cm² ±0.3 cm²
Acoustic gel for general use: pH = 5.5-8
Acoustic resistance: $\dot{y}$ 1.7*10⁵ g/cm² s
Working mode: continuous wave Doppler
Audio output
Audio output power: <0.5W
Audio port: $\dot{y}$ 3.5mm/USB interface
Battery: 9V lithium battery (IEC type 6F22 or equivalent)
Standby: >4 h
Suitable time in pregnancy: 16 weeks or later
Size: 104.5*50*70mm
Weight: 69.5g (without battery)

Conditions

Working temperature: 5-40°C
Working humidity: 0-80% RH
Atmospheric pressure: 86-106 kPa
Storage temperature: -20-55°C
Storage humidity: 10-93% RH
Atmospheric pressure: 50-106 kPa

Troubleshooting

The following problems may occur while using the device:

Difficulties	Cause	Solution
Creaking sound	High volume, lots of gel on the skin, low battery	Turn down the volume, use less gel, replace the battery
Weak audio output	Low volume, insufficient gel, low battery	Increase volume, use more gel, replace battery
Low sensitivity	Wrong probe position, insufficient amount of gel	Find a suitable position, add gel

Schematic representation of use

1. Lie down comfortably.
2. Connect the headphone jack and put on the headphones.
3. Move the device over the abdomen and locate the position of the fetal heart.
4. Listen to the heartbeat.

Distributor
Sunnysoft sro
Kovanecka 2390/1a
190 00 Prague 9
Czech Republic
www.sunnysoft.cz

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